VERMEIREN

Express VB

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

EN

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Instructions to the specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: B, 2016-08

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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© Vermeiren GROUP 2016

Contents

1	Product description.....	3
1.1	Intended Use	3
1.2	Technical specifications	4
1.3	Components	6
1.4	Accessories	6
1.5	Explanation of symbols	6
1.6	Safety instructions	7
2	Use	8
2.1	Remarks on electromagnetic compatibility (EMC).....	8
2.2	Carrying the electric wheelchair.....	9
2.3	Assembly and disassembly of the electric wheelchair	9
2.4	Fold up or unfolding the wheelchair.....	10
2.5	Mounting or removing backrest	11
2.6	Mounting or removing battery housing	11
2.7	Mounting or removing armrest.....	12
2.8	Mounting or removing of the footrests.....	13
2.9	Operating the parking brakes.....	13
2.10	Transfer in and out the electric wheelchair.....	14
2.11	Correct position in the electric wheelchair	14
2.12	Driving the electric wheelchair.....	14
2.13	Operating the electromagnetic brakes	15
2.14	Driving the electric wheelchair on stairs	18
2.15	Driving the electric wheelchair on ramps.....	18
2.16	Pushing the wheelchair	19
2.17	Transport in the car	20
2.18	Transport in an airplane	20
2.19	Batteries	21
2.20	Battery charger.....	21
2.21	Charging the batteries	21
2.22	Thermal fuse	22
3	Installation and adjustment.....	22
3.1	Tools	22
3.2	Manner of delivery	22
3.3	Possible adjustments.....	23
3.4	Battery connections	24
3.5	Changing the batteries	25
3.6	Tyre changing	25
4	Maintenance	27

Preface

First of all we want to thank you for putting your trust in us by selecting one of our products.

The expected lifetime of your electric wheelchair is strongly influenced by the care and maintenance of the wheelchair.

This manual will help you get acquainted with the operation of your electric wheelchair.

Following of the user instructions and the maintenance instructions are an essential part of the warranty.

This manual reflects the latest product developments. Vermeiren has the right to introduce changes without the obligation to adapt or replace previously delivered models.

For any further questions, please consult your specialist dealer.

1 Product description

1.1 Intended Use

The electric wheelchair Express VB is equipped with two motors, of 150W.

The electric wheelchair Express VB is intended to comfortably transport persons with walking difficulties or no walking abilities. This electric wheelchair is designed to transport 1 person.

The electric wheelchair is designed to transport persons only, no goods.

Neither should it be used by persons obviously suffering from physical or mental limitations (e.g. visual impairment, ...) that render them incapable of safely handling the electric wheelchair.

The different types of fittings and accessories, and the modular construction allow full use by persons disabled by:

- paralysis
- loss of limbs (leg amputation)
- limb defects or deformations
- stiff or damaged joints
- heart insufficiencies and poor blood circulation
- balance disturbances
- cachexia (decrease in muscle)

The electric wheelchair shall not be used by persons that are in a condition that they can fall out the electric wheelchair, or use for these patients safety belts.

The electric wheelchair is classified as class A.

The electric wheelchair is suited for indoor and limited outdoor use by the occupant.

The user can operate the wheelchair by himself or have the wheelchair pushed by an attendant with the handgrips.

When providing for individual requirements:

- body size and weight (max. 90 kg)
- physical and psychological condition
- residential circumstances
- environment

should be taken into consideration.

Your electric wheelchair should be used on flat surfaces where all four wheels are touching the ground and where there is sufficient contact to operate the wheelchair safely.

You should practice for use on uneven surfaces (cobblestones, etc.), slopes, curves and to get past obstacles (curbs, etc.). A particular risk is involved when crossing surfaces such as ice, grass, rubble, foliage, etc.

The electric wheelchair should not be used as a ladder, nor is it a transport for heavy or hot objects.

When using your electric wheelchair on streets or footpaths, local laws and regulations apply.

The electric wheelchair can be used on sidewalks, urban road. Under no circumstances the electric wheelchair may be used for driving on larger roads or expressways.

Use only Vermeiren approved accessories.

The manufacturer is not liable for damage caused by the lack of improper service or as a result of not following instructions from this manual.

Visually impaired people can contact the dealer for the instructions for use.

1.2 Technical specifications

Technical terms below are valid for the electric wheelchair in standard settings and optimum environmental conditions. If accessories are used, the tabulated values will change. Changes in outdoor temperatures, humidity, uphill's, downhill's, soils and battery levels can reduce output.

Output:

Brand	Vermeiren				
Address	Vermeirenplein 1/15, B-2920 Kalmthout				
Type	Electric wheelchair, Class A				
Model	Express VB				
Maximum occupant mass	90 kg				
Description	Dimensions				
Effective seat width	390 mm	420 mm	440 mm	460 mm	500 mm
Overall width (depends on the seat width)	580 mm	610 mm	630 mm	650 mm	690 mm
Description	Dimensions Minimum		Dimensions Maximum		
Max. speed	6 km/h				
Continuous driving distance range*	Approx. 18 km (Battery 38 Ah) Approx. 25 km (Battery 50 Ah)				
Overall length with footrest	950 mm				
Overall height (backrest included)	960 mm				
Folded / dismantled length (without footrests, battery housing)	790 mm				
Folded / dismantled width (wheelchair folded)	400 mm				
Folded / dismantled height (back folded, without armrests)	710 mm (for seat width 460 mm; depends on seat width)				
Total mass	From 67 kg (For seat width 390 mm; depends on seat width and any accessories)				
Mass of heaviest part (that can be dismantled or removed)	Frame, armrest + joystick, seat cushion, backrest cushion: From 29,95 kg				
Masses of parts that can be dismantled or removed.	Armrest without joystick: 1,60 kg; Footrests: 1,85 kg; Battery housing: 15,90 kg/piece (Batteries: 13,5 kg/piece); Backrest: 1,80 kg				
Static stability downhill	7,5°				
Static stability uphill	12°				
Static stability sideways	10°				
Max. safe slope	7°				
Obstacle climbing	15 mm				
Ground clearance	30 mm				
Seat plane angle	5°				
Effective seat depth	430 mm				
Thickness seat cushion	40 mm				
Seat surface height at front edge (with seat cushion)	540 mm				
Backrest angle	5°				
Backrest height	430 mm				

Brand	Vermeiren	
Address	Vermeirenplein 1/15, B-2920 Kalmthout	
Type	Electric wheelchair, Class A	
Model	Express VB	
Maximum occupant mass	90 kg	
Description	Dimensions Minimum	Dimensions Maximum
Distance between footrest and seat	400 mm	520 mm
Angle between footrest and footplate	80°	100°
Angle between seat and footrest	105°	
Angle between seat and footplate	5°	25°
Distance between armrest and seat (with seat cushion)	225 mm	315 mm
Front location of armrest structure	400 mm	
Driving motors	2 x 150W + Electromagnetic braking system	
Batteries	2 x 12V --- AGM / 38 Ah or 50 Ah / 20 hr	
Battery charger	See separate manual battery charger (Use ONLY the battery charger that is delivered with your electric wheelchair)	
Thermal fuse	30 AMP	
Operator control	See separate operator control	
Minimum turning diameter	Approx. 1400 mm	
Reversing width	Not applicable	
Diameter Rear wheels (number)	12 1/2 x 2 1/4" air (2)	
Tyre pressure, rear wheels **	Max. 2,5 bar	
Diameter front wheels (number)	8 x 1 3/4 mm air (2)	
Tyre pressure, front wheels **	Max. 2,5 bar	
Noise level	< 65 dB (A)	
Storage and use temperature	-20 °C to +40 °C	
Operating temperature of the electronics	-10 °C to +40°C	
Storage and use humidity	30%	70%
We reserve the right to introduce technical changes. Measurement tolerance ± 15 mm / 1,5 kg / ° . * The theoretical driving distance will be reduced if the wheelchair is used frequently on slopes, rough ground or to climb curbs. **Since different tyres may be used, please note the correct operating pressure of the tyres you use. For other tyre diameters refer to your specialist dealer.		

Table 1: Technical specifications

The wheelchair complies to the requirements set up in:


ISO 7176-8: Requirements and test methods for static, impact and fatigue strengths.

ISO 7176-14: Wheelchairs - Part 14: Power and control systems for electrically powered wheelchairs and scooters -- Requirements and test methods

EN 1021-1: Furniture - Assessment of the ignitability of upholstered furniture - Part 1 : Ignition source smouldering cigarette

EN 1021-2: Furniture - Assessment of the ignitability of upholstered furniture - Part 2 : Ignition source match flame equivalent

1.3 Components

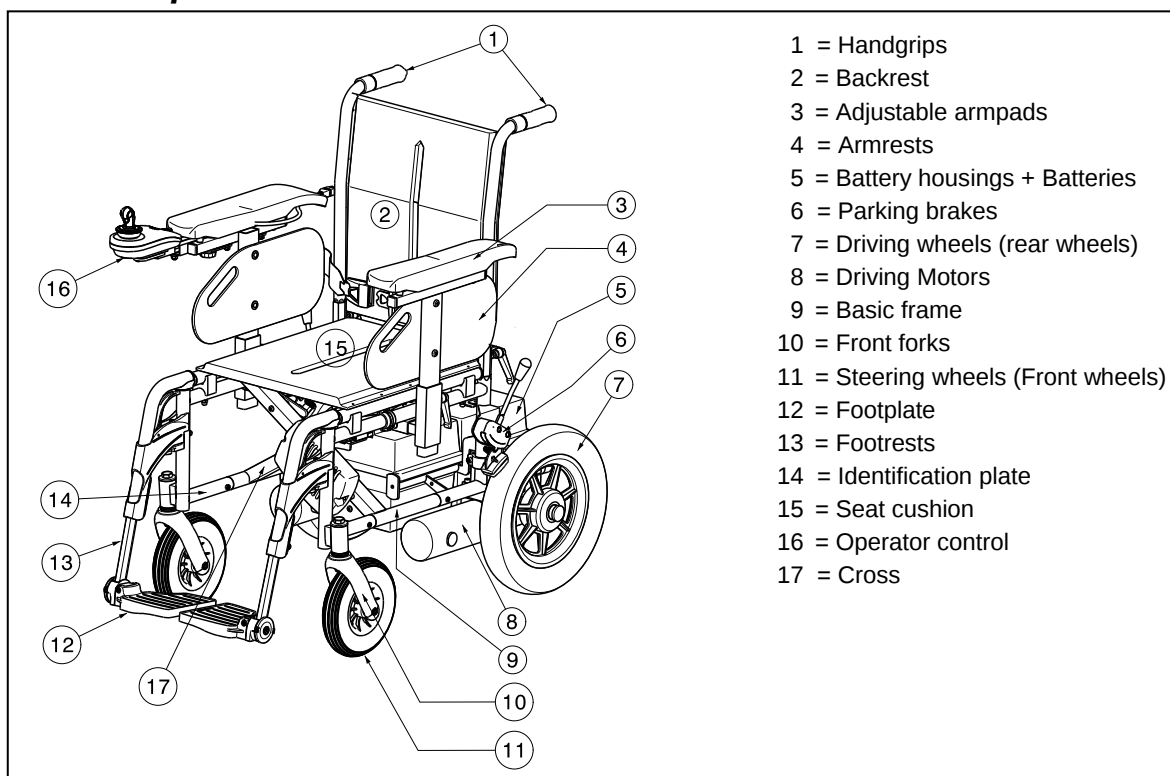1.4 Accessories

The following accessories are available for the electric wheelchair Express VB:

- Back inclination (SE52)
- Formed backrest and seat (L24 / L25)

1.5 Explanation of symbols

Maximum mass

Indoor and limited outdoor use

Indoor use (only for battery charger)

Separate recovery and recycling of electric and electronic devices (only for battery charger)

Protection class II

Maximum safe slope

CE conformity

Maximum speed
km/h

Position: Parking brakes activated (electric driving possible)

Position: Parking brakes deactivated (free running and pushing possible, no electric driving)

During free running, be careful with slopes and inclinations

Not intended to be used as a seat in a Motor vehicle

Type designation

1.6 Safety instructions

- ⚠ To prevent injury and/or damage to your wheelchair, make sure that no objects and/or body parts are caught in the rims of the driving wheels.
- ⚠ Follow the instructions for using your wheelchair. For example, avoid driving without brakes against an obstacle (step, edge of the curb) or dropping down from steps.
- ⚠ When getting into and out of the wheelchair, do not stand on the footplates. The footplate should be folded up beforehand.
- ⚠ Investigate the effects of shifting the centre of gravity on the behavior of the wheelchair, for example on up or down gradients, on laterally sloping ground, or when overcoming obstacles. Obtain support from an attendant.
- ⚠ If you want to pick up something (lying in front of, on the side, or to the rear of the wheelchair), you should not lean too far out to avoid tipping over.
- ⚠ When moving through doors, arches, etc. ensure that there is enough room at the sides so that you do not get your hands or arms caught or crushed and that there is no damage to the wheelchair.
- ⚠ Avoid uncontrolled rolling against obstacles (steps, curbs, doorframes, etc) or dropping down from ledges. The manufacturer cannot assume liability for damage caused by overloading, collision or other improper use.
- ⚠ When moving in public places, you are subject to local regulations.
- ⚠ Influence of medicine or alcohol reduces your driving capabilities.
- ⚠ When travelling outdoors, adapt your driving to weather and traffic conditions.
- ⚠ Do not use your electric wheelchair in rainy conditions.
- ⚠ Do not put your electric wheelchair into the free-wheeling position on slopes.
- ⚠ Never reverse uphill.
- ⚠ Reduce speed when you go round corners.
- ⚠ To be better visible when driving in the dark, wear the brightest possible clothing or clothes with reflectors and make sure that the reflectors of the wheelchair are clearly visible and drive with lightning on.
- ⚠ Make sure that the lights and reflectors of your electric wheelchair are free from dirt and/or other objects that could hide them.
- ⚠ When transporting the wheelchair, never pick it up by grasping movable parts (wheels, arm supports, footrests, etc.)
- ⚠ When transporting the electric wheelchair, no persons may be transferred along with it.
- ⚠ No additional passengers may be taken.
- ⚠ When storing or parking your electric wheelchair outside, protect it with a shrouding cover against humidity.
- ⚠ High levels of humidity or very cold conditions can reduce the performance of your electric wheelchair.
- ⚠ Never use your electric wheelchair as a seat in an automobile or other vehicle.

- ⚠ Never exceed the maximum load of 90 kg.
- ⚠ Use only Vermeiren approved accessories.
- ⚠ Before getting on or off, taking apart or transporting your electric wheelchair, put your wheelchair off with "ON/OFF" button.
- ⚠ Check that the profile depth of the tyres is adequate.
- ⚠ If your electric wheelchair has pneumatic tyres, be careful to inflate them to the correct pressure (*see pressure indication on the tyres*).
- ⚠ Stairs may only be negotiated by an attendant. If devices and furnishings like ramps or lifts are available, use them.
- ⚠ Risk of injury in case of a sudden stop - Always use the safety belt.
- ⚠ Use restraining safety belts for patients that makes extreme movements so the wheelchair can tip over.

2 Use

This chapter describes the everyday use. **These instructions are for the user and the specialist dealer.**

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

The electric wheelchair is delivered fully assembled by your specialist dealer. The instructions intended for the specialist dealer for adjusting the electric wheelchair are given in § 3.

2.1 Remarks on electromagnetic compatibility (EMC)

Your electric wheelchair has been tested according to EN 61000-4-3 and EN 61000-4-2 for EMC-compliance.

We wish to point out that sources of electromagnetic waves are liable to create interferences. The electronics of the wheelchair itself could affect other electric appliances too.

To reduce the effect of electromagnetic sources of interference, please read the following warnings:

- ⚠ **WARNING:** The wheelchair might disturb the operation of devices in its environment that emit electromagnetic field.
- ⚠ **WARNING:** The driving performance of the wheelchair can be influenced by electromagnetic fields (e.g. electricity generators or high power sources).
- ⚠ **WARNING:** Avoid using any portable TV or radio in the immediate environment of your wheelchair for as long as it is turned on.
- ⚠ **WARNING:** Avoid using any transmitter-receiver in the immediate environment of your wheelchair for as long as it is turned on.
- ⚠ **WARNING:** Check the area for transmitter masts and avoid using the wheelchair close to them.
- ⚠ **WARNING:** If involuntary movements or braking occur, turn off the wheelchair as soon as it is safe to do so.

Interfering electromagnetic fields may have a negative effect on the wheelchair's electronic systems. These can include:

- Disengagement of the engine brake
- Uncontrollable behavior by the wheelchair
- Unintentional steering movements

In the presence of very strong or enduring interfering fields, the electronic systems could even break down entirely or suffer permanent damage.

Possible sources of radiation include:

- Portable receiver and transmitter installations (receiver and transmitter with fixed antenna)
 - Transmitting and receiving sets
 - Portable TV, radio and navigation devices
 - Other personal transmitting devices
- Mobile medium-range transmitting and receiving devices (e.g. car antennas)
 - Fixed transmitting and receiving sets
 - Fixed mobile transmitting and receiving devices
 - Fixed radio, TV and navigation systems
- Long-range transmitting and receiving devices
 - Radio and television towers
 - Amateur radio sets
- Other home devices
 - CD player
 - Notebook
 - Microwave oven
 - Cassette recorder
 - etc.

Devices like electric shavers and hair dryers will have no influence provided that they function perfectly and their cabling be in an excellent condition. Please conform to the operating instructions accompanying such electric instruments to ensure trouble free operating of your wheelchair.

2.2 Carrying the electric wheelchair

The best way to carry the electric wheelchair is to make use of free-wheel mode of the electric wheelchair. Place the wheelchair in free-wheel mode and roll the wheelchair with the handgrips to the desired place.

When the wheelchair must be taken up or down stairs, do this with the wheelchair ramps or lift systems. At least two persons are required to move the wheelchair up or down stairs or over single steps.


Another way to carry the electric wheelchair is to dismantle the electric wheelchair. Carry the separate parts (frame, armrests, footrests, battery housings, backrest) to the desired place. Grasp the wheelchair only on fixed frame parts (never by the footrests, armrests and wheels).

2.3 Assembly and disassembly of the electric wheelchair

The electric wheelchair is delivered fully assembled. Your dealer delivers the wheelchair fully assembled and explains the various operating elements and their use. However, for your own safety we provide a further, detailed explanation of the different parts.

2.3.1 Assembly of the electric wheelchair

⚠ CAUTION: Risk of clamping – Check that no cables can be squashed or squeezed.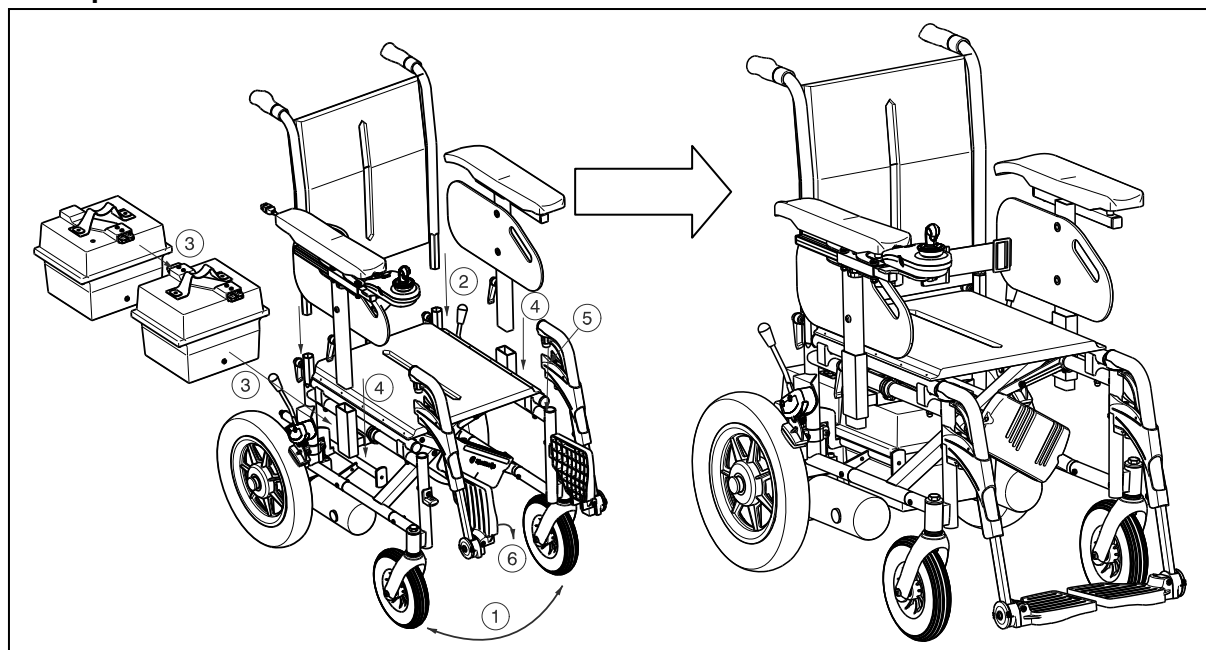

To assembly the electric wheelchair:

1. Unfold the wheelchair (paragraph 2.4).
2. Mount the backrest (paragraph 2.5).
3. Mount the 2 battery housings (paragraph 2.6).
4. Mount the armrests (paragraph 2.7).
5. Mount the footrests (paragraph 2.8).
6. Fold the footplates of the footrests down.

To disassembly the electric wheelchair do the reverse of the above instructions.

2.4 Fold up or unfolding the wheelchair

⚠ CAUTION: Risk of clamping – Keep fingers, buckles, clothes away from moving parts of the wheelchair (eg. cross, ...).

To unfolding the wheelchair:

1. Position yourself behind the wheelchair.
2. Use the hand-grips to open the wheelchair as much as possible.
3. Position yourself at the front of the wheelchair.
4. Push both seating tubes down till they are fixed in their position.
5. Place the battery housings on the guide rails.


To fold up the wheelchair:

1. Remove the battery housings.
2. Fold or remove the footplates (see § 2.8).
3. Take the seat on the front side and backside and pull it up.

2.5 Mounting or removing backrest

- ⚠ WARNING:** Risk of injury – Make sure that both securing levers ① are engaged.
- ⚠ CAUTION:** Chance of pinching – Do not place fingers between the backrest tubes of the wheelchair.

For transport the backrest of the wheelchair can be completely removed and put on the seat.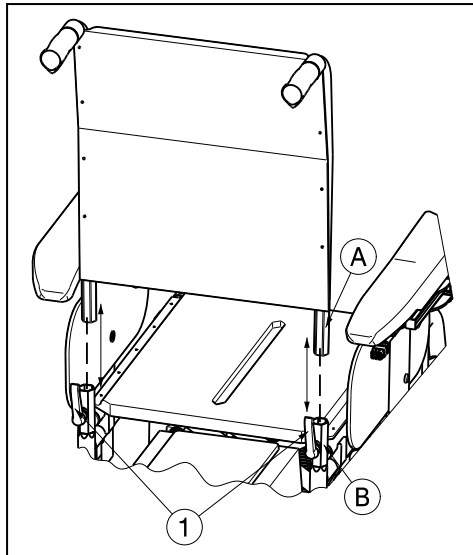

To mount the backrest on the electric wheelchair:


1. Check that the levers ① are not fixated otherwise loosen the levers ①.
2. Mount the backrest tubes ① in the hexagonal tubes of the frame ②.
3. Retighten the levers ① properly.

To remove the backrest from the electric wheelchair:

1. Turn on the levers ① until they come loose.
2. Remove the backrest tubes ① out the hexagonal tubes of the frame ②.
3. Put the backrest on the seat for transportation.

2.6 Mounting or removing battery housing

- ⚠ CAUTION:** Chance of pinching – Do not place fingers between the battery housings and guide rails of the wheelchair.

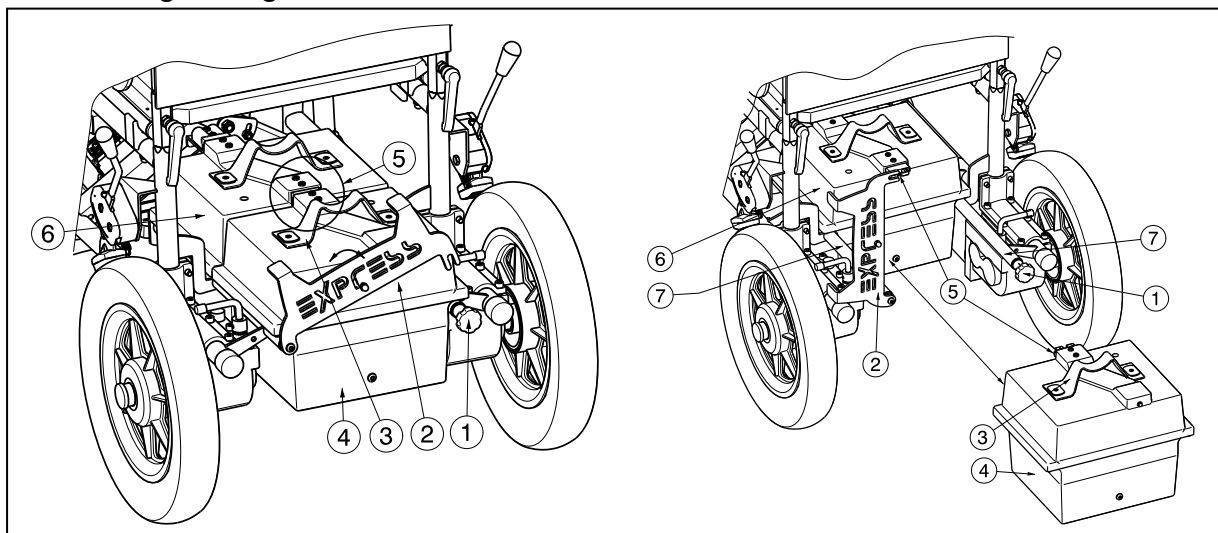To mount the battery housings on the electric wheelchair:

1. Grasp the handgrip ③ of the first battery housing (without thermal fuse) ⑤.
2. Lift the battery housing ⑤ on the side rails ⑦ of your wheelchair.
3. Push the battery housing ⑤ as much as possible under the seat of the wheelchair.
4. Check that the connectors (battery, power module) fits in each other.
5. Grasp the handgrip ③ of the battery housing (with thermal fuse) ④.
6. Lift the battery housing ④ on the side rails ⑦ of your wheelchair.
7. Push the battery housing ④ backwards.
8. Check that the battery connectors ⑤ fits in each other.
9. Turn the plate ② downwards.
10. Retighten the star knob ①.


To remove the battery housings of the electric wheelchair:

1. Loosen the star knob ①.
2. Rotate the plate ② upwards and to the left.
3. Grasp the handgrip ③ of the first battery housing (with thermal fuse) ④.
4. Slide the battery housing backwards until the battery connectors ⑤ come loose.
5. Lift the first battery housing ④ out the wheelchair.
6. Slide the second battery housing ⑥ backwards until you can lift the housing out the wheelchair frame.
7. Lift the second battery housing ⑥ out the wheelchair.

2.7 Mounting or removing armrest

- ⚠ **WARNING:** Risk of injury and/or damage - Make sure that the levers ① are properly tightened before using the wheelchair.
- ⚠ **WARNING:** Risk of tipping over - Check that the armrests are at the same height on each side.
- ⚠ **CAUTION:** Risk of clamping – Keep fingers, buckles and clothes away from the attachment points of the armrests.

The armrests can be removed so the patient can be moved sideways. The armrests can also be removed for therapeutic purposes and to move the patient from the wheelchair.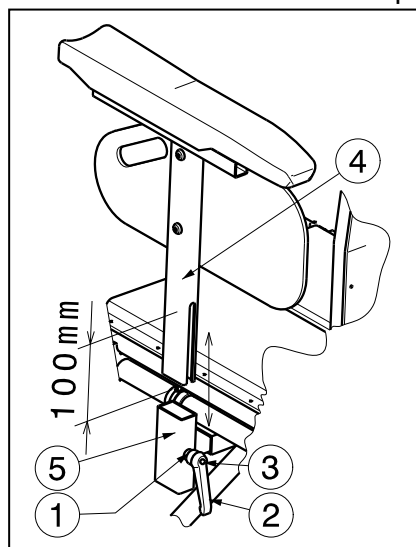

To mount the armrests on the electric wheelchair:

1. Mount the armrests ④ in the rectangular tube ⑤.
2. Retighten the handle ① properly. (If the lever of handle ② is not in a good position, press on button ③ on the handle and place the lever ② in a good position that no risk of injury to the hands can appear.)

To remove the armrests from the electric wheelchair:


1. Turn on the handle ① until it come loose. (If the lever of handle ② is not in a good position, press on button ③ on the handle and place the lever ② in a good position that no risk of injury to the hands can appear.)
2. Remove the armrests ④ out the rectangular tube ⑤.

The armrests are adjustable in different heights over a range of 90 mm (stepless). Keep a safe distance from 10 mm on the bottom of the armrest.

1. Loosen the lever ① a little.
2. Move the rectangular tubes ④ and ⑤ over each other until the desired armrest height.
3. Retighten the lever ① hand-tightened.
4. Check that the lever ① is securely fixed.
5. Adjust the other armrest with the same instructions.

2.8 Mounting or removing of the footrests

⚠ CAUTION: Risk of injury – Make sure that the footrests are firmly fixated.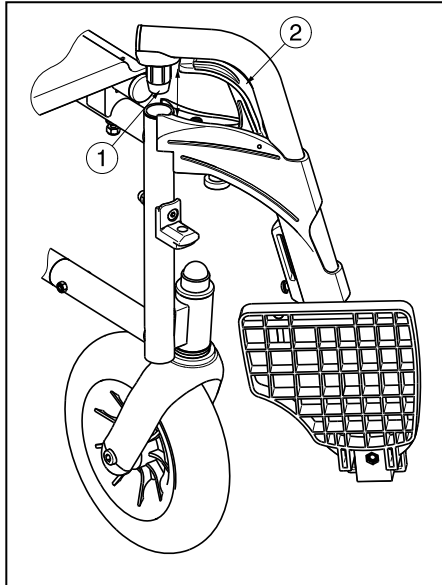

The mounting of the footrests is done as follows:

1. Hold the footrest sideways at the outside of the wheelchair frame and mount the tube hood ① into the frame.
2. Swing the footrest inwards till it clicks in position.
3. Swing the foot plate downwards.
4. Check that the footrest is well fixated.

To take off the footrests:

1. Pull handle ②.
2. Swing the footrest to the outside of the wheelchair till it comes loose from the guidance.
3. Pull the footrest from tube hood ①.


2.9 Operating the parking brakes

For each drive wheel your electric wheelchair has standard parking brakes.

⚠ WARNING: The brakes are not used to slow down the wheelchair during movements – Use the brake only to prevent the wheelchair from unintended movements.

⚠ WARNING: Good operation of the brakes is influenced by wear and contamination of the tires (water, oil, mud, ...) – Check the condition of the tires before each use.

⚠ WARNING: The brakes are adjustable and can wear – Check the operation of the brakes before each use.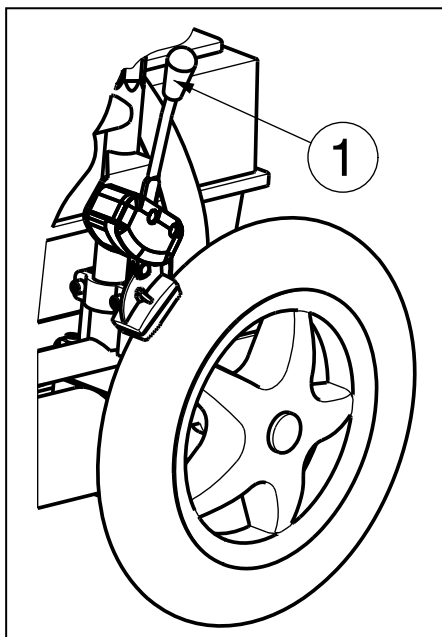

To apply the brakes:

1. Push the brake levers ① forward till you feel a distinctive click.

⚠ CAUTION: Risk of unintended movement – Make sure the wheelchair is on a flat horizontal surface before releasing the brakes. Never release both brakes simultaneously.


To release the brakes:

1. Release one brake by pulling the lever ① backwards.
2. Release the second brake by pulling the lever backwards.

2.10 Transfer in and out the electric wheelchair

- ⚠ **WARNING:** Power down the wheelchair before getting in or out the wheelchair.
- ⚠ **CAUTION:** In case you cannot perform the transfer in a safe manner, ask someone to assist you.
- ⚠ **CAUTION:** Risk of injury – Do not use the joystick, footplates, armrests as support.
- ⚠ **CAUTION:** Risk of tipping over of the wheelchair – Do not stand on the footplates.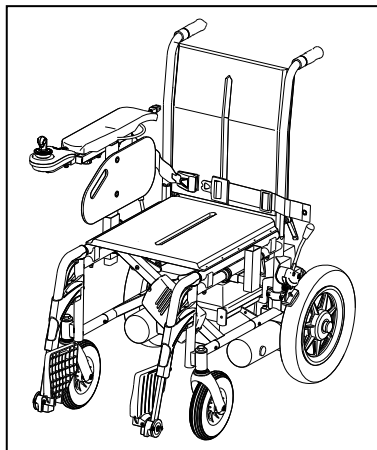

1. Park the electric wheelchair as close as possible to the place where you want to transfer.
2. Check that the operator unit is switched off. Make sure that the wheelchair is NOT in freewheel mode.
3. Fold the footplates upwards to prevent standing on them.
4. If the transfer is on the side of the wheelchair, remove the armrest on that side.
5. Transfer to / from the electric wheelchair.

2.11 Correct position in the electric wheelchair

Some recommendations for a comfortable use of the electric wheelchair:

1. Position your backside as close as possible to the backrest.
2. Make sure your upper legs are horizontal – If needed adjust the length of the footrests.

2.12 Driving the electric wheelchair

- ⚠ **WARNING:** Risk of burns – Be careful when driving in hot or cold environments (sunshine, extreme cold, etc.) for a sufficient amount of time and when touching - Surfaces can assume the environment temperatures.
- ⚠ **WARNING:** Risk of unsafe settings - Use only the driving characteristics described in this manual.

2.12.1 Preparing the electric wheelchair for use

When you start using your electric wheelchair, make sure that it is standing on level ground. All wheels must be in contact with the ground.

1. Make sure that the freewheel mode is OFF and the operator unit is switched off.
2. Adjust the wheelchair best suited for yourself.
3. Set the operator unit in the most comfortable position.
4. Sit down on the seat and verify that both armrests have been adjusted so your forearms are folded down.
5. Put your wheelchair on with the "ON/OFF" button from the operator unit.

Now put the speed control on the operator unit to the minimum position. Your electric wheelchair is now ready for use.

2.12.2 Handling after usage

Before getting off from your electric wheelchair, make sure that all four wheels touch the ground simultaneously.

Push on the "ON/OFF" button from the operator unit, the display of the operator unit goes out.

2.13 Operating the electromagnetic brakes

Let go of the joystick to stop the electric wheelchair.

2.13.1 Parking the electric wheelchair

Once your electric wheelchair has been turned off, no command can be sent to the driving system. Always park your electric wheelchair on sites that are easy accessible and on horizontal surfaces that the four wheels are touching the ground.

2.13.2 Operator units

The instructions for use from the operator units you can find in the separate user manuals of the operator units that is included with your wheelchair.

Changing the software is only allowed by Vermeiren. For changes in the software contact Vermeiren.

2.13.3 Adjusting operator unit


⚠ CAUTION: Risk of pinching - Do not put your fingers between the operator control and the other parts when positioning the operator unit.

The operator unit's horizontal position can be changed:

1. Loosen screw ① just under the armrest a little bit.
2. Move the operator unit to the desired position or remove the operator unit.
3. Retighten screw ① properly.


If you pull screw ① to the side, the operator unit can be turned to the side.

NOTE: Depending on the features of your wheelchair, screw ① could also be located beneath the guide tube ②.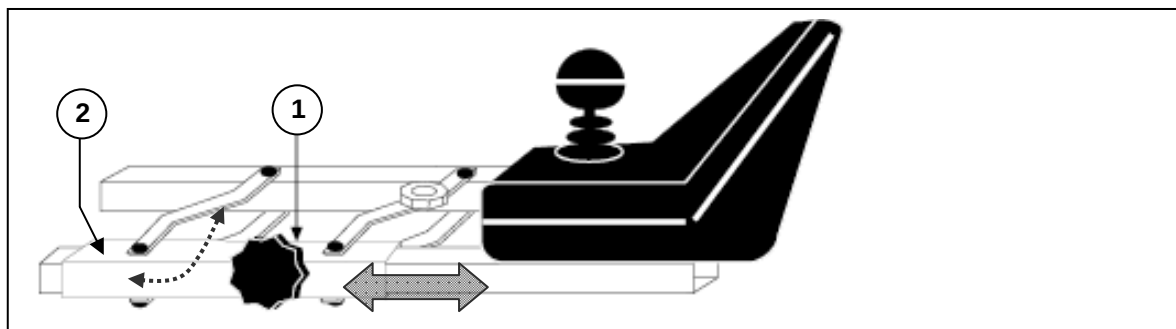

2.13.4 Connections for charger / program unit

⚠ WARNING: Risk of injury - Check that all plugs (battery charger and programming unit) are removed before setting the wheelchair in motion.

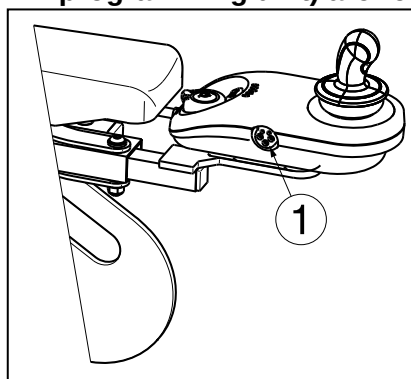The socket ① for connecting the charger supplied is located on the front of the operator unit. Here, too, is the connection for the programming unit, which should only be connected and used by authorised persons who have been trained to program it (Vermeiren personnel).

Make sure that the joystick is in the neutral (central) position when pressing the on/off button, otherwise the electronics will be blocked. This block can be lifted by switching the control unit off and then on again.

2.13.5 Your first trip

⚠ WARNING: Control your electric wheelchair - Accustom yourself with the driving behavior of your electric wheelchair.

- Driving

Once you are seated on your electric wheelchair and have started it as described above, take the joystick with your hand and push this in the required direction i.e.:

PUSH ACTION FORWARD	=	FORWARD MOVEMENT
PUSH ACTION BACKWARD	=	BACKWARD MOVEMENT

- Braking

To brake, let go of the joystick, which will let it return to the zero position and slow down your electric wheelchair to a gentle stop. Practice pulling away and braking to get accustomed to the electric wheelchair. You need to be able to estimate how your electric wheelchair will react when you drive or brake.

- Driving in corners and bends

⚠ WARNING: Risk of tipping over - Reduce your speed before you enter a curve or corner.

⚠ WARNING: Risk of clamping - Always maintain an adequate distance from corners and obstacles.

Move the joystick into the position you wish to turn. The front wheels will turn accordingly and steer the electric wheelchair into a new direction. It is very important that you ensure that there is enough space allowing you to go around bends and corners. Narrow passages must preferably be approached in a large curve so as to allow you to enter the narrowest part as straight from the front as possible.

Take care not to enter curves and corners diagonally. By 'cutting the corner', chances are that your rear wheels, back of wheelchair will run into obstacles and destabilize your electric wheelchair.

2.13.6 Backward movement

⚠ WARNING: Control your electric wheelchair - Accustom yourself with the driving behavior of your electric wheelchair.

⚠ WARNING: Control your speed - Always go backward in the lowest speed possible.

⚠ WARNING: Risk of collision - Always look to the back when you go backward.

Backward driving requires increased concentration and care. This explains why we have greatly reduced the speed of backward movement in comparison with forward driving. But we still recommend that you put the speed on minimum when you drive backward.

Left and right controls act inverse when driving backwards.

2.13.7 Going uphill

- ⚠ WARNING: Control your electric wheelchair - Accustom yourself with the driving behavior of your electric wheelchair.**
- ⚠ WARNING: Control your electric wheelchair - Never put your electric wheelchair in freewheel on slopes.**
- ⚠ WARNING: Control your speed - Move on slopes as slow as possible.**
- ⚠ WARNING: Risk of tipping over - Do not exceed the maximum gradient of static and dynamic stability uphill (see paragraph "Technical specifications").**
- ⚠ WARNING: Never reverse going uphill.**

Always approach slopes directly from the front and, to avoid tipping over, see that all four wheels stay in contact with the ground at all times (ramps, driveways, etc.).

If you come to a stop on a slope by letting go of the joystick, the engine break will prevent your electric wheelchair from rolling backward. As soon as the joystick returns to zero position, the engine break is activated.

To resume your uphill drive, push the joystick as much as possible to the front to ensure the release of a sufficient amount of power. This will allow your electric wheelchair to slowly ascend the slope.

If your wheelchair is unable to drive up, turn the speed control up and try again.

2.13.8 Going downhill

- ⚠ WARNING: Control your electric wheelchair - Accustom yourself with the driving behavior of your wheelchair.**
- ⚠ WARNING: Control your wheelchair - Never put your electric wheelchair in neutral on slopes.**
- ⚠ WARNING: Control your speed - Move on slopes as slow as possible.**
- ⚠ WARNING: Risk of tipping over - Avoid sharp bends.**
- ⚠ WARNING: Risk of tipping over - Do not exceed the maximum gradient of static and dynamic stability downhill (see paragraph "Technical specifications").**

Always approach downhill slopes directly from the front. Slantwise approaches can lead to some of the wheels no longer staying in contact with the ground (danger of tipping over).

The weight of the electric wheelchair will increase your downhill speed. Lower the speed on the operator unit.

Avoid sharp bends on downhill slopes. The weight of your electric wheelchair could cause your electric wheelchair to lift up on one side or even fall over around bends.

2.14 Driving the electric wheelchair on stairs

- ⚠ WARNING: Risk of injury - Sit quietly in your wheelchair and avoid jerky movements. If you cannot sit quietly when being transported because of illness or disability (e.g. slightly spastic, etc.), then at least two persons are required to secure the wheelchair on both sides by taking hold of fixed parts of the frame.**
- ⚠ WARNING: Risk of injury – Wear suitable safety belt systems while being transported (information obtainable from your specialist dealer).**
- ⚠ WARNING: Risk of injury – Make sure that your arms and legs do not protrude outside the wheelchair while being driving on stairs.**

If you want to be taken up or down stairs with the wheelchair, you will need a wheelchair ramps or wheelchair liftsystems.

2.15 Driving the electric wheelchair on ramps

- ⚠ WARNING: Risk of injury - Do not exceed the maximum load of the ramps.**
- ⚠ WARNING: Risk of injury - Choose the correct ramps to prevent injury or damage.**
- ⚠ WARNING: Risk of injury - Be sure that the wheel height is high enough to get over the free height of the ramps. The electric wheelchair frame may not touch the ramps.**
- ⚠ WARNING: Risk of injury - Use a restraining safety belt to secure yourself in your wheelchair.**
- ⚠ WARNING: Risk of tipping over - Put the adjustment functions (seat, backrest, footrests, ...) so that the wheelchair has the best stability.**

Should you wish to use ramps to drive over an obstacle, please note the following:

1. Find out from the manufacturer what the maximum load is for the ramps.
2. Drive on the ramps at the lowest speed possible.
3. See the instructions in the chapter "your first trip".

If the wheelchair has adjustment functions, ensure that (because these has influence on the stability of the electric wheelchair):

1. The seat is in the lowest position as possible and in the horizontal position.
2. The backrest is in the upright position.
3. The footrests have been set so that no collision can occur while passing the obstacle.

If another person is pushing you, note that the large weight of the electric wheelchair exerts appreciable reverse forces.

2.16 Pushing the wheelchair

- ⚠ **WARNING:** Control your electric wheelchair - Never put your electric wheelchair in neutral while you are driving.
- ⚠ **WARNING:** Control your electric wheelchair - Never put your electric wheelchair in neutral on slopes. It may accidentally roll away.


The wheelchair is fitted with a freewheel device that is accessible and operable only by the attendant.

2.16.1 150W motors - 6km/h

Put the chair in neutral with the adjusting levers ① on both motors. These levers are located on the back of the electric wheelchair.

Clutch or declutch the motor by following the instructions below:

- 
DRIVING
 Pull the adjusting levers ① of both motors in the driving position. Loosen the parking brakes. Switch the operator unit out and in. Electronically controlled driving is now possible.
- 
NEUTRAL
 Push the adjusting levers ① of both motors in the neutral position and you will activate the wheelchair's free-running mode. The wheelchair can now be pushed without electronic drive.
- 
 In free wheel mode, the electronic brake is deactivated and the wheelchair will no longer be held back. Do not activate free wheel mode on slopes.

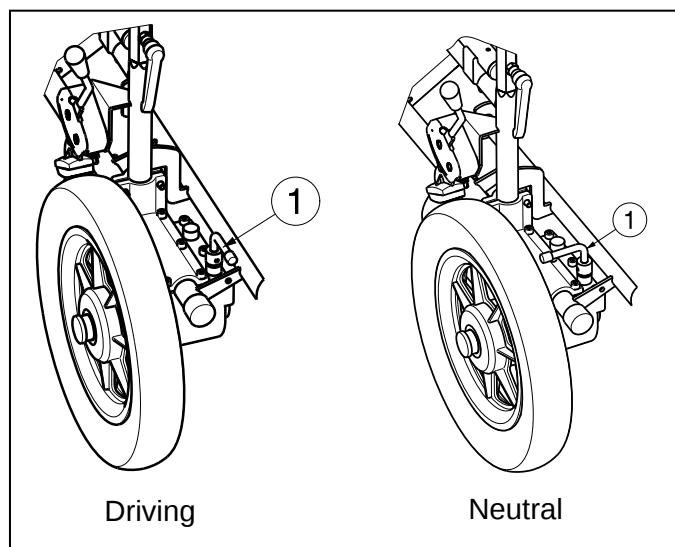- ⚠ When free-running is activated, the electromagnetic braking system is deactivated, so the wheelchair is no longer secured against rolling away. Take care not to set the wheelchair down on slopes or uneven terrain, or it may accidentally roll away.
- ⚠ The electronic system will indicate, through the lock symbol flashing, that electronic driving is not possible.

2.17 Transport in the car

- ⚠ **DANGER:** Risk of injury – The wheelchair is not suited for use as a seat in a motor vehicle.
- ⚠ **WARNING:** Risk of injury - No people or objects should be under the electric wheelchair, during transportation.
- ⚠ **WARNING:** Risk of injury - See that the wheelchair is attached properly. So you can avoid injury from the passengers during collision or sudden braking. Make sure that they are only attached to solid parts of the frame.
- ⚠ **WARNING:** Risk of injury- Use for attaching the wheelchair and passenger **NEVER** the same seatbelt.

Never use your wheelchair as a seat in an automobile or other vehicle. The wheelchair shall be marked with following symbol.

To transport the wheelchair in the car use following steps:

The best way to transport your electric wheelchair in the car is to drive the electric wheelchair in the car by using ramps.

When you are not experienced to drive the electric wheelchair by using of ramps you can also put the wheelchair in neutral mode and push the wheelchair in the car by using ramps.

When the wheelchair does not fit in the car it is also possible to transport the wheelchair by the following steps:

1. Remove all moveable parts prior to transporting (footrests, armrests, battery housings etc.).
2. Store moveable parts safely.
3. Place the electric wheelchair in the car with 2 persons by grasping to the fixed parts of the frames.
4. Attach the solid parts of the frame securely to the vehicle.
5. Place the wheelchair in drive mode (engage the parking brakes) and check that the operator unit is switched off.

2.18 Transport in an airplane

The electric wheelchair can be transported in an airplane. Electric wheelchairs shall be checked at the desk. Announce the wheelchair on time by the concern airline before departure. Do not use the electric wheelchair as a seat in an airplane, the wheelchair must be stored in the cargo room.

To transport the wheelchair in an airplane note the following requirements:

1. Type and properties of wheelchair (joystick, batteries)

The wheelchairs shall be foreseen of dry or gel batteries. Mostly these batteries must not be removed out the wheelchair. Only disconnect and insulate the electrical connections on the battery.

2. Dimensions and weight of the wheelchair

The allowable weight and dimensions of the wheelchair depends on the type of the airplane.

3. Damage on the wheelchair

It is possible that the wheelchair could be damaged because the wheelchair will be stored in a narrow area with the suitcases and other goods.

To prevent damage on the wheelchair:

- Put the electrical adjustments in standard position (lift column as low as possible, seat inclination horizontal, back inclination the most forward).
- Fold the footplates upwards, place the armrests in the most inward position.
- Check that the adjustment levers shall positioned inwards.
- Cover the operator unit with an soft material that can withstand shocks.

Before travelling contact the concerned airline about the requirements for transporting the wheelchair in an airplane.

2.19 Batteries

The standard for your electric wheelchair are two closed AGM batteries, 12 V/38 Ah or 50 Ah. The batteries used with your electric wheelchair are drive batteries which only attain full capacity after a few charging and use cycles.

If the batteries loose their power after long usage, or if they are damaged, get them both replaced by a specialist dealer only.

We accept no liability for damage caused through using other types of batteries.

If the batteries are opened, all manufacturer liability and all claims will become void.

Do not use the batteries at temperatures below -5°C or above +50°C (the ideal is: +20°C).

Use only the original batteries. We do not accept any liability for damages resulting from use of other batteries.

2.20 Battery charger

To charge the batteries, only use the battery charger supplied.

For the user instructions of the battery charger you can refer to the manual that is delivered with your battery charger.

2.21 Charging the batteries

⚠ CAUTION: Risk of injury - Only use the battery charger supplied to charge batteries.

As the charger aligns the charge curve with the batteries charge level, you can recharge your wheelchair after each use. This avoids any aggressive battery charging and the "memory effect".

Recharge the wheelchair, at the latest, when the charge indicator on the operator unit goes into the red zone. If, despite this, you continue driving, eventually only the last red diode lights up and flashes continually, indicating that the batteries are nearly flat. If you disregard this warning signal, too, an error message will shortly appear indicating that the batteries can no longer provide power for driving. The batteries should therefore be charged before these error messages appear, using the supplied battery charger. Avoid the batteries becoming drained, in any case.

For the use of the battery charger refer to the separate manual of the battery charger.


If you decide not to use your electric wheelchair for an appreciable period, you must nevertheless recharge it regularly to keep it in a running condition ready for immediate use.

The manufacturer accepts no liability for damage caused by improper charging.

For further information, please refer to the user instructions provided with the charger.

2.22 Thermal fuse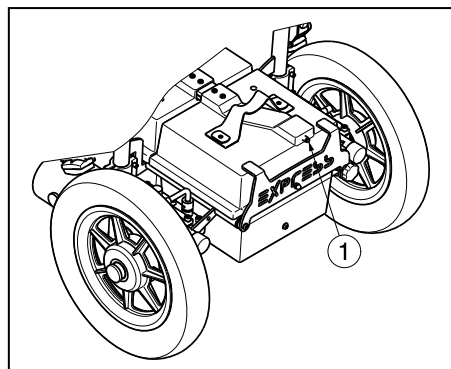

To protect the motor against overload, on the back of the wheelchair topside of the first battery housing, there is a thermal safety mechanism ① that will automatically cut off the motor to prevent overheating and thus rapid wear and tear or breakdowns. This can occur if you go up or down slopes that exceed the maximum gradient indicated. Nominal loads exceeding the maximum could also trigger the safety mechanism.

To be able to use the electric wheelchair again, remove the overload and wait till the motor has cooled down. Then gently press the thermal safety mechanism back in. Your wheelchair is now ready for use again.

3 Installation and adjustment

The instructions in this chapter are for the specialist dealer only.

The Express VB has been designed as a rear wheel drive electric wheelchair.

- ⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.**
- ⚠ WARNING: Risk of tipping over - Variation allowed adjustments can still change the stability of your wheelchair (tilt back or sideways).**

3.1 Tools

To set up the wheelchair the following tools are needed.

- Wrench set n° 10
- Allen keyset n° 4 to n° 5

3.2 Manner of delivery

The electric wheelchair shall be delivered with:

- Framework with armrests, operator unit, rear and front wheels, seat and backrest
- 1 pair of footrests
- 2 separate drive batteries + battery housings, 2 x drive motors
- Battery charger + manual battery charger
- Electronics
- Tools
- Operator unit + manual operator unit
- Accessories


Before use, check that everything is included and that no products are damaged (example by transport, ...). Please note that the basic configuration may differ in different European countries. Contact your specialist dealer.

3.3 Possible adjustments

- ⚠ **WARNING:** Risk of injury - Turn the electric wheelchair off before you perform any adjustments.
- ⚠ **WARNING:** Risk of injury - Never perform any adjustments while you are driving.
- ⚠ **WARNING:** Risk of injury - Adjustments may only performed by your specialist dealer.
- ⚠ **WARNING:** Risk of injury - Make sure all screws, levers are firmly secured before driving with your electric wheelchair.
- ⚠ **CAUTION:** Risk of clamping – Keep fingers, buckles and clothes away from the attachment points or any moving parts.

3.3.1 Adjusting the armrests

The width of the armrests can be adjusted in horizontal direction.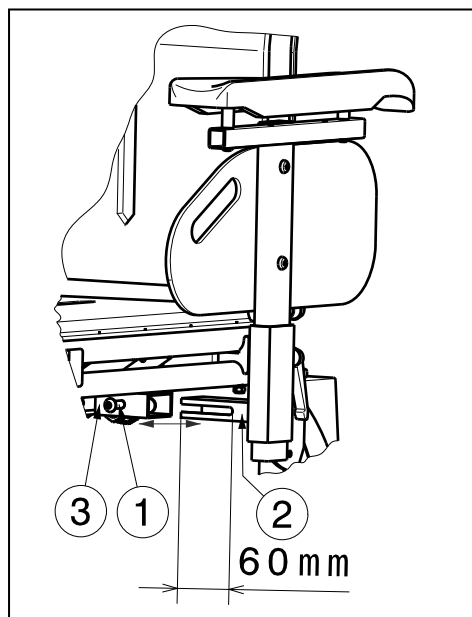


1. Loosen the hexagon socket set screws ① just under the seat frame.
2. Move the armrest in an horizontal direction until the correct position, by moving the rectangular tubes ② and ③ over each other. (Range 60 mm: step less)
3. Retighten the screws ① properly.
4. Repeat instructions above for the other armrest.

Check that the armrests are positioned identically on both sides.

3.3.2 Adjusting of the footrests

* Adjusting length:

- ⚠ **CAUTION:** Risk of damage – Avoid that the footrests make any contact with the ground. Keep a minimum distance from 30 mm above the ground.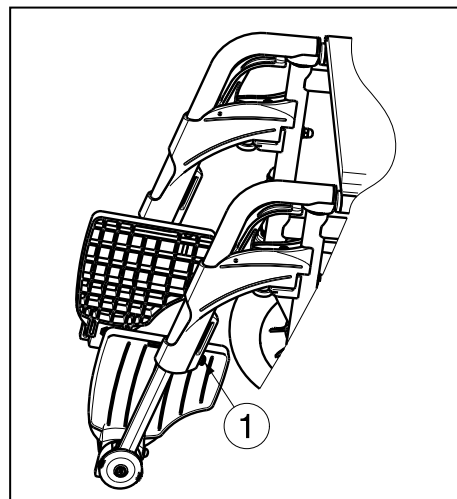

Adjust the length of the footrests as follow:

1. Remove the screw ① (on the back of the footrests).
2. Adjust the length of the footrest to a comfortable length (range: 120 mm, stepless).
3. Tighten the screw ① properly.


* Adjusting the footplates:

The footplates can be used in 2 positions. Standard they are adjusted with the base plate in the back position.

The footplates can be adjusted to the front by switching the left and right footplate.

* Adjusting the footplates: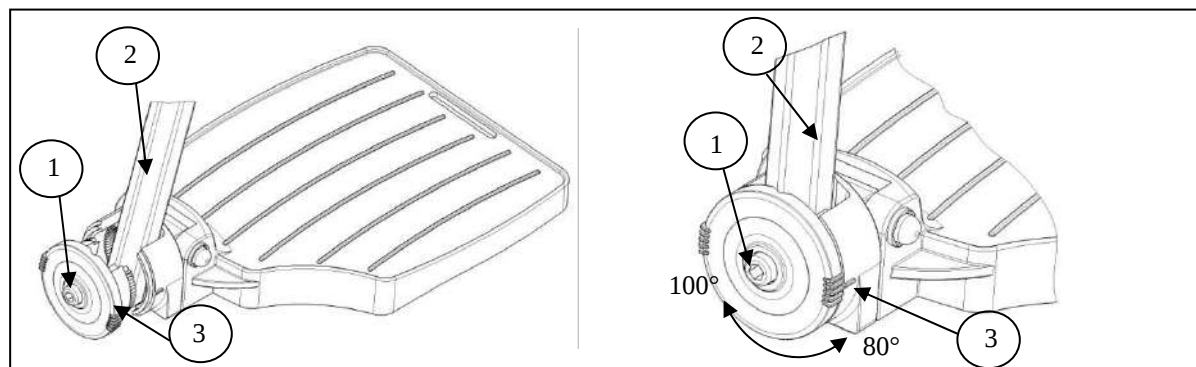

Adjust the angle of the footplates as follow:

1. Loosen bolt ①.
2. Adjust tube ② in the desired angle. The angle indication (80°-85°-90°-95°-100°) is according the dashes ③.
3. Retighten the bolt ①.

3.4 Battery connections


⚠ CAUTION: Risk of injury - Check that no tools or other conductive objects come between the batteries poles, to avoid unintentionally current flows.

⚠ CAUTION: Risk of injury, damages - Do not work in a humid environment with batteries and electronics.

The best way to disconnect the batteries:

1. Disconnect battery 1.
2. Disconnect battery 2.


To connect the batteries use the reverse order.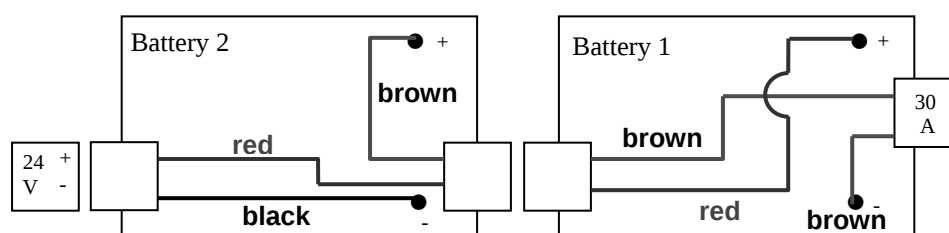

3.5 Changing the batteries

⚠ CAUTION: Risk of burns – Do not come in contact with the acid from the batteries. See for a good ventilation of the battery compartment.

The battery shall be changed by trained personnel.

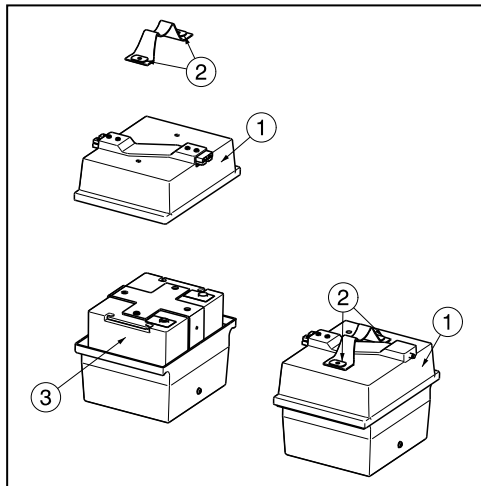The battery can be changed according following steps:

1. Remove the two battery housings from the wheelchair.
2. Remove the battery cover ① by loosen the bolts ② on the handgrip ③.
3. Disconnect all the wires on the batteries.
4. Lift the batteries ③ out the battery housing.
5. Place the new batteries in the battery housing.
6. Connect all the wires to the batteries.
7. Replace the battery cover ① with the bolts ②.

3.6 Tyre changing

⚠ CAUTION: There must be no air in the tube before the tyre can be removed.

⚠ CAUTION: Risk of damage - If handled improperly, the rim might become damaged.

Only an expert can guarantee correct assembly. Work not done by your specialist dealer, would void any warranty claims.

Use only inflating equipment which complies to regulations and indicates the pressure in bar. We do not accept any liability for damage caused by using inflation equipment or wheels not supplied by the manufacturer.

⚠ WARNING: Risk of injury - Check that the pressure is correct.

⚠ CAUTION: Risk of injury - Make sure that no objects, body parts or inner tubes are pinched between the tyre and the rim when mounting a tyre.

- **Steering wheels (front wheels)**

Removing tyre from rim:

1. Let all the air out of the tube.
2. Insert a tyre lever between the tyre and the rim.
3. Slowly and carefully push the lever downwards. This will pull the tyre over the edge of the rim.
4. Move the lever along the rim, the tyre will jump out.
5. Now carefully remove the tyre from the rim and then remove the tube.

Note the following before inserting the new tube:

Check the rim bed and the inside wall of the tyre for foreign matter and clean these properly if necessary. Check the condition of the rim bed, especially around the position of the air valve. Please use only genuine original replacement parts. No liability is accepted for damage caused by non-genuine replacement parts. Kindly contact your specialist dealer.

Assembly tyre from rim:

Put the deflated tube around the center of the rim. Make sure that the valve juts out the valve opening in the rim.

Push the tyre over the edge of the rim, starting behind the air valve. Inflate the tube slightly until it is round, and place it inside the tyre.

If the tube fits snugly inside the tyre without any folds (in the case of folds: let out some air), then the upper side of the tyre can be pressed lightly onto the rim with both hands, starting at the air valve.


Check all around on both sides that the tube is not pinched between the rim and the edge of the tyre. Lightly push the air valve inwards and pull it out again to make sure that the tyre is positioned properly in the region of the air valve.

To ensure that the wheel is inflated correctly, admit only so much air initially that the tyre can still be easily pushed inwards by using your thumbs. If the check-lines are equidistant from the edge of the rim on both sides of the tyre, then the tyre is centered properly. If not - let out the air and position the tyre afresh. Now the tyre can be inflated to its full operating pressure (note the maximum) and the valve cap should be replaced.

When inflating the tyres, always check that the pressure is correct. The correct pressure is given on the tyre sides.

- **Driving wheels (rear wheels)**

⚠ WARNING: Risk of injury - Make sure all screws are firmly secured by hand before driving with your electric wheelchair.

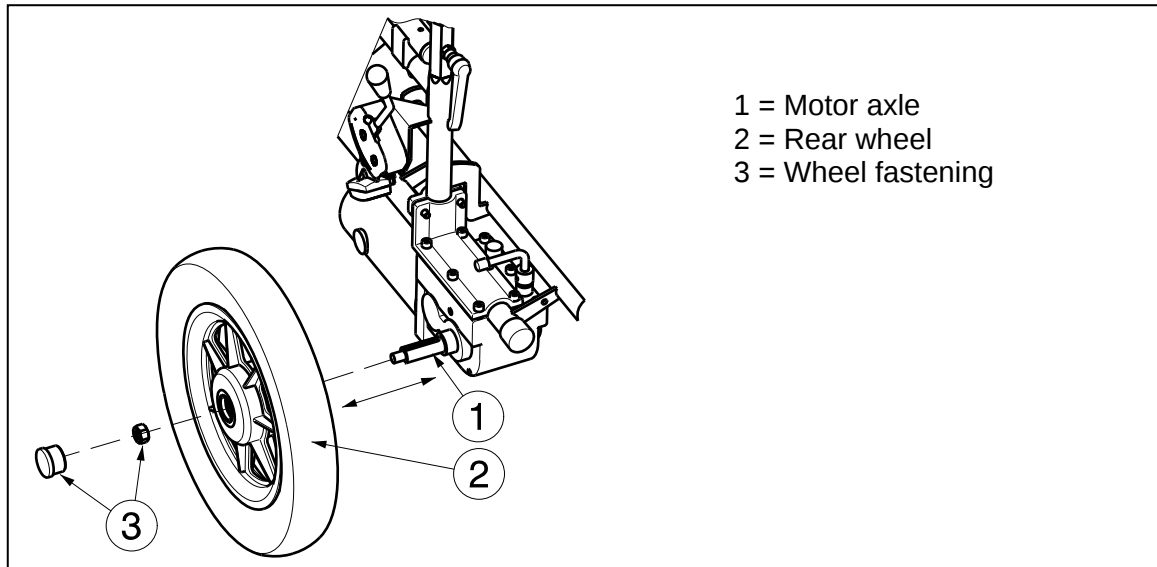DISMANTLING

1. Loosen and remove the wheel fastenings ③.
2. Take the complete rear wheel ② from the motor axle ①.

ASSEMBLY

1. Make the threads and Motor axle free from dirt and grease the threads.
2. Mount the rear wheel on the motor axle so that the wheel hub fits on the motor axle groove.
3. Secure the wheels firmly with the nut (wheel fastenings ③).

4 Maintenance

For the maintenance / cleaning instructions refer to the separate maintenance manual of the electric wheelchairs. Refer for these separate manual to your specialist dealer.

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ENGLISH

CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim tis warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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VERMEIREN GROUP
Vermeirenplein 1/15
B-2920 Kalmarhout
BELGIUM

Notes

A series of horizontal dotted lines for taking notes.

Notes

A series of horizontal dotted lines for taking notes.

Belgium

N.V. Vermeiren N.V.

Vermeirenplein 1 / 15
B-2920 Kalmthout
Fax: +32(0)3 666 48 94
website: www.vermeiren.be
e-mail: info@vermeiren.be

France

Vermeiren France S.A.

Z. I., 5, Rue d'Ennevelin
F-59710 Avelin
Tel: +33(0)3 28 55 07 98
Fax: +33(0)3 20 90 28 89
website: www.vermeiren.fr
e-mail: info@vermeiren.fr

Italy

Vermeiren Italia

Viale delle Industrie 5
I-20020 Arese MI
Tel: +39 02 99 77 07
Fax: +39 02 93 58 56 17
website: www.vermeiren.it
e-mail: info@vermeiren.it

Poland

Vermeiren Polska Sp. z o.o

ul. Łączna 1
PL-55-100 Trzebnica
Tel: +48(0)71 387 42 00
Fax: +48(0)71 387 05 74
website: www.vermeiren.pl
e-mail: info@vermeiren.pl

Czech Republic

Vermeiren ČR S.R.O.

Nádražní 132
702 00 Ostrava 1
Tel: +420 596 133 923
Fax: +420 596 121 976
website: www.vermeiren.cz
e-mail: info@vermeiren.cz

Germany

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Wahlerstraße 12 a
D-40472 Düsseldorf
Tel: +49(0)211 94 27 90
Fax: +49(0)211 65 36 00
website: www.vermeiren.de
e-mail: info@vermeiren.de

Austria

Vermeiren Austria GmbH

Schärdinger Strasse 4
A-4061 Pasching
Tel: +43(0)7229 64900
Fax: +43(0)7229 64900-90
website: www.vermeiren.at
e-mail: info@vermeiren.at

Switzerland

Vermeiren Suisse S.A.

Eisenbahnstrasse 62
3645 Gwatt (Thun)
Tel: +41(0)33 335 14 75
Fax: +41(0)33 335 14 67
website: www.vermeiren.ch
e-mail: info@vermeiren.ch

Spain / Portugal

Vermeiren Iberica, S.L.

Carratera de Cartellà, Km 0,5
Sant Gregori Parc Industrial Edifici A
17150 Sant Gregori (Girona)
Tel: +34 972 42 84 33
Fax: +34 972 40 50 54
website: www.vermeiren.es
e-mail: info@vermeiren.es

Manufacturer

Vermeiren GROUP

Vermeirenplein 1 / 15
B-2920 Kalmthout
Tel: +32(0)3 620 20 20
Fax: +32(0)3 666 48 94
website: www.vermeiren.be
e-mail: info@vermeiren.be

VERMEIREN

Maintenance manual
Electric wheelchair

Contents

1	Maintenance	2
1.1	Regular Maintenance	2
1.2	Shipping and storage	3
1.3	Care.....	4
1.4	Inspection.....	5
1.5	Troubleshooting	6
1.6	Disinfection	6
2	Warranty	8
3	Disposal.....	8
4	Declaration of conformity	8
5	Maintenance plan.....	9
6	Disinfection book.....	9

1 Maintenance

⚠ CAUTION: Risk of electrical shocks - Disconnect the batteries to prevent any unwanted current flow before performing any maintenance work.

Lifetime of the electric wheelchair is strongly influenced by its use, storage, regular maintenance, servicing and cleaning.

The batteries are maintenance-free.

1.1 Regular Maintenance

The maintenance activities to ensure a good condition for your Vermeiren wheelchair are described below:

- Before every ride
 - Check the tyres for visible damage and/or soiling. Remove any dirt as it could impair the motion of the tyres. When a tyre is damaged, please go see an authorized workshop for repair.
 - Check the tyre pressure and inflate them if necessary (only for air tyres).
 - Check the motor brakes, parking brakes (if available) and replace if necessary.
 - See that all the screws are tightened properly.
 - Check the electric wheelchair (cleanliness, cracks, damage on structural parts ...) and make it clean. If necessary change the coating.
 - Switch on the electronics after verifying that the motor is engaged. The operator control indicates the level of the batteries. If the battery charge is not sufficient for the intended route, switch off the operator control and charge the batteries.
 - Use the drive electronics display to check that the driving, braking and adjustment features are fully-functional. If these functions are impaired, contact your dealer.
- Every 8 weeks: inspection and possibly lubricating or adjusting from
 - Armrests, footrests or footplates, wheel axles
 - Dirt under the battery cover (carefully remove any dirt as it can corrode the plugs).
 - ⚠ **WARNING: Remove the pole connectors before cleaning the poles of the batteries.**
 - ⚠ **WARNING: Risk of Short circuit - Be careful not to put any objects capable of conducting an electric current between the poles of the batteries.**
 - Dirt / corrosion on the poles of the batteries (be careful to keep the poles of the batteries clean as dirt may damage the batteries).
 - Tighten moving parts that can be removed.
 - Condition of the cushioning materials
 - Profile depth of the wheels

- Every 6 months or for every new user
 - General review
 - Mechanical integrity
 - Cleanliness, Disinfection
 - Function of the wheels and check tyre pressure
 - That the charger is working
 - Check the following greasing points:
 - a) All movable parts

For your convenience, on the back of this manual is a maintenance plan.

Repairing and assembling of spare parts for your wheelchair may only be performed by the specialist dealer.

Only authorized Vermeiren spare parts may be installed.

1.2 Shipping and storage

The shipping and storage of the electric wheelchair shall be according following instructions:

- Store in a dry place (between +5 °C and +45 °C).
- The relative humidity of the air should be between 30% and 70%.
- Provide sufficient covering or packaging to protect the electric wheelchair from rust and foreign bodies. (e.g. salt water, sea air, sand, dust).
- Store all removed parts together in one place (or mark them if necessary) to avoid mixing up with other products when re-assembling (e.g. the charger).
- Components must be stored without being subjected to strains (Do not put too heavy parts on the electric wheelchair, not clamping between something, ...). Grasp the wheelchair by solid frame parts only.
- Check internal cables for squashing and the prevention of kinks.
- Disconnect the power plug of the battery charger.

If batteries are not used, they can completely drain. So if you no longer use your electric wheelchair, you could leave it connected to the battery charger. Charging is automatically controlled by the battery charger. If you remove and store the batteries, kindly note the following:

- Remove the cable clamps from the poles of the battery.
- The positive pole must at least be covered by a pole cap.
- Only grasp the batteries by two opposite sides of the unit.
- Make sure that no objects can make contact with both poles during the time of storage (danger of short circuits!).
- Batteries should only be stored in dry, well-ventilated spaces at a temperature between 0°C and +40°C. Preferred storage temperature: **+20°C**).
- Keep the batteries in the battery boxes for protection against moisture or other external influences.
- Protect the plugs and sockets of the battery boxes against corrosion.
- Protect the batteries against deep discharging (see the chapter on battery charging).

If you have further questions, consult your specialist dealer who will gladly help you in regard to storage and servicing of your batteries.

1.3 Care

The cleaning of your wheelchair is possible with household products.

1.3.1 Seat and back cushion

The cleaning of the seat and back cushion shall be done as follows:

- Clean the seat and back cushion with a cloth moistened in warm water. Take care that you do not soak the cover.
- Use a mild commercial detergent for removing stubborn dirt.
- Stains can be removed by using a sponge or a soft brush.
- Do not use strong cleaning liquids like solvents, nor use hard brushes.
- Never clean with steam and/or pressure washers.

Damage caused by the use of improper cleaning agents is not the responsibility of Vermeiren.

1.3.2 Plastic parts

Clean plastic parts of your electric wheelchair with commercial plastic cleaners. Only use a soft brush or soft sponge. Take note of special product information.

Examples of plastic parts are the footplates, battery cover, etc.

1.3.3 Coating

The high quality of the surface layer guarantees optimal protection against corrosion. If the outer coating is damaged by scratching or in some other way, get your specialist dealer to repair the affected surface.

When cleaning, only use warm water and normal household detergents and soft brushes and cloths. Ensure that no moisture gets into the components (tubes, electronics, ...).

Initially the chrome parts only require rubbing with a dry cloth. Dull places or stubborn dirt can best be removed by using a suitable commercial chrome polish.

Dulling can be avoided by lightly greasing steel parts with Vaseline.

1.3.4 Electronics

⚠ WARNING: Check the plug connectors for corrosion or damage before every ride, since it could affect the efficiency of the electronics.

⚠ WARNING: Remove the batteries before performing any maintenance, to avoid involuntary conduction of currents.

Wipe the operator control with a cloth moistened by a few drops of a commercial domestic cleaner. Do not use any abrasives or sharp-edged polishing equipment like a metal scrubber or brush, since these can scratch the surface of the operator control.

1.4 Inspection

In principle we recommend one inspection every year, and a minimum of one before usage is resumed. All of the following checks must be performed and documented by authorized persons:

- Visually check the frame parts for plastic deformation, cracks and impaired functioning. (basic frame, seat frame, back frame, side parts, footrests, legrests, motor suspensions).
- Visually check all plastic parts for cracks and brittle spots.
- Check the solidity and seating of all screws.
- Visually check all housings for damage, screws must be securely fixed, seals and gaskets should not exhibit visible damage.
- Visually check for damage to the paintwork (danger of corrosion).
- Check the plug connectors for corrosion or damage, since it could affect the efficiency of the electronics.
- Check the operation of the tyres (free running, level rolling, axle play, tyres, profile, condition of the rims, air pressure in the case of air tyres, floating axles, etc.).
- Verify the amount of grease on the metal joints of movable parts.
- Check the functioning of the armrests, footrests and legrests (locking, load, deformation, wear caused by loads).
- Check the operation of other detachable parts (example: personal safety belt, battery housing, fitting back/seat, etc.).
- Check the cabling (especially for: crushing, abrasion, cuts, visible insulation of the inner conductor, visible metallic veins, kinks, lumpiness, color changes of the outer sleeve, brittle spots).
- Electric leads to be securely placed to avoid shearing, crushing and other mechanical stresses and strains.
- For the charger of the electric wheelchair measure the resistance of the protective braid (O) according to VDE 0702-1.
- For the charger of the electric wheelchair measure the residual diversion current (A) according to VDE 0702-1.
- For the charger of the electric wheelchair measure insulation resistance (MO) according to VDE 0702-1.
- Check the functioning of the drives (during a test drive → noises, speed, free running, etc.), if necessary: Measure the performance, first with no load and then with the nominal load ("SWL"), to check for wear and tear of the motors by comparing the values of the electric current with the values when the electric wheelchair was delivered.
- Check the motors for adjustments (seat inclination, backrest inclination, lift column).
- Check the condition of the batteries, covers and tubes.
- Remove carbon dust of carbon brushes and if necessary replace carbon brushes.
- Completeness of the delivery condition, instruction manual available?

Higher performance and/or high utilization (daily use or use on difficult terrain), increase the inspection intervals for wear parts (carbon brushes, pollution motor, tyre profile, ...) to at least half-yearly review.

Checking measurements may only be carried out by skilled persons trained on the wheelchair and at least under the supervision of an electrician who knows the checking instruments and processes. Only an electrician is able to release the electric wheelchair for use after performing measurements or servicing.

The service must only be confirmed in the maintenance plan if a minimum of all the above-mentioned aspects have been checked.

1.5 Troubleshooting

The following list will assist you with the troubleshooting of your electric wheelchair.

Problem	Cause of problem
The electric wheelchair refuses to go once started. No indication of charge level	• Electric wheelchair is not switched on. • Battery plug not connected (batteries not making contact). • Thermal safety fuses disengaged. (check that the fuses of the operator control and the batteries are defective or have burn out). • Batteries defective (totally discharged). • Defective operator control. • Loom of cables defective.
The electric wheelchair refuses to go once started. Charge level insufficient.	• Motor/gears in neutral. • Magnetic brake defective. • Motor defective.
Operating error	• Possible safety shut down or operating error (joystick). • Put the joystick in neutral position and restart the system.
Thermal safety fuses disengage.	• Motor overloaded (see "Technical specifications"). • Thermal safety fuses defective.
Battery error	• Check batteries and cabling, if necessary charge. • If necessary, replace batteries.
Batteries cannot be charged.	• Batteries not fitted correctly. • Loading socket defective. • Wrong charger. • Loading device defective.

1.6 Disinfection

⚠ WARNING: Dangerous Products - The use of disinfectants is restricted to authorized personnel.

⚠ WARNING: Dangerous Products, risk of skin irritation - Wear suitable protective clothing. Take note of the product information of the solutions concerned.

All parts of the wheelchair can be treated by scrubbing with a disinfectant.

All steps taken to disinfect rehabilitation equipment, their components or other accessory parts are to be recorded in a disinfection report containing a minimum of the following information (with product documentation appended):

Date of the disinfection	Reason	Specification	Substance and concentration	Signature
--------------------------	--------	---------------	-----------------------------	-----------

Table 1: Example of a disinfection book

Abbreviations used in column 2 (reason):

V = Suspected infection IF = Infection case W = Repetition I = Inspection

For a blank sheet of a disinfection book see § 6.

The recommended disinfectants for scrubbing (based on the list provided by the Robert Koch Institute, RKI) are standing in the table below. The current state of the disinfectants included in the RKI list can be obtained from the Robert Koch Institute (RKI) (homepage: www.rki.de).

Active substance	Product name	Laundry disinfection		Surface disinfection (scrubbing-/wiping disinfection)		Disinfection of excretions 1 part sputum or stools + 2 parts diluted solution or 1 part urine + 1 part diluted solution						Area of effectiveness	Manufacturer or Supplier	
						Sputum		Stools		Urine				
		Diluted solution	Time to take effect	Diluted solution	Time to take effect	Diluted solution	Time to take effect	Diluted solution	Time to take effect	Diluted solution	Time to take effect			
		%	Hr.	%	Hr.	%	Hr.	%	Hr.	%	Hr.			
Phenol or phenol derivative	Amocid	1	12	5	6	5	4	5	6	5	2	A	Lysoform	
	Gevisol	0,5	12	5	4	5	4	5	6	5	2	A	Schülke & Mayr	
	Helipur			6	4	6	4	6	6	6	2	A	B. Braun	
	m-cresylic soap solution (DAB 6)	1	12	5	4							A		
	Phenol	1	12	3	2							A		
Chlorine, organic or inorganic substances with active chlorine	Chloramin-T DAB 9	1,5	12	2,5	2	5	4					A ¹ B		
	Clorina	1, 5	12	2,5	2	5	4					A ¹ B	Lysoform	
	Trichlorol	2	12	3	2	6	4					A ¹ B	Lysoform	
Per combinations	Apesin AP100 ²			4	4							AB	Tana PROFESSIONAL	
	Dismozon pur ²			4	1							AB	Bode Chemie	
	Perform ²			3	4							AB	Schülke & Mayr	
	Wofesteril ²			2	4							AB	Kesla Pharma	
Formaldehyde and/or other aldehydes or derivatives	Aldasan 2000			4	4							AB	Lysoform	
	Antifect FD 10			3	4							AB	Schülke & Mayr	
	Antiseptica surface disinfection 7			3	6							AB	Antiseptica	
	Apesin AP30			5	4							A	Tana PROFESSIONAL	
	Bacillocid special			6	4							AB	Bode Chemie	
	Buraton 10F			3	4							AB	Schülke & Mayr	
	Desomed A 2000			3	6							AB	Desomed	
	Hospital disinfectant cleaner			8	6							AB	Dreiturm	
	Desomed Perfekt			7	4							AB*	Desomed	
	Formaldehyde-solution (DAB 10), (formaline)	1,5	12	3	4							AB		
	Incidin Perfekt	1	12	3	4							AB	Ecolab	
	Incidin Plus			8	6							A	Ecolab	
	Kohrsolin	2	12	3	4							AB	Bode Chemie	
	Lysoform	4	12	5	6							AB	Lysoform	
	Lysoformin	3	12	5	6							AB	Lysoform	
	Lysoformin 2000			4	6							AB	Lysoform	
	Melsept	2	12	4	6							AB	B. Braun	
	Melsitt	4	12	10	4							AB	B. Braun	
	Minutil	2	12	6	4							AB	Ecolab	
	Multidor			3	6							AB	Ecolab	
	Nüscosept			5	4							AB	Dr. Nüsken Chemie	
		Optisept			7	4							AB*	Dr. Schumacher
		Pursept-FD			7	4							AB*	Merz
		Ultrasol F	3	12	5	4							AB	Fresenius Kabi
Amphoteric surfactants (amfotensiden)	Tensodur 103	2	12									A	MFH Marienfelde	
Lye	Lime-milk ³							20	6			A ³ B		
1 Not effective against myco-bacteria when service disinfecting, especially in the presence of blood. 2 Not suitable for disinfecting blood-contaminated or porous surfaces (e.g. raw wood). 3 Useless for tuberculosis; preparation of Lime-milk: 1 part dissolved lime (calcium hydroxide) + 3 parts water. * Checked for effectiveness on viruses in accordance with checking methods of the RKI (Federal Health Reporting 38 (1995) 242). A: Suitable for killing vegetative bacterial germs including myco-bacteria as well as fungi, including fungal spores. B: Suitable for deactivating viruses.														

Table 2: Disinfectants

Kindly consult your specialist dealer if you have queries on matters related to disinfection; he will gladly assist you.

2 Warranty

Excerpt from the "General Business Conditions":

(...)

5. The terms of the guarantee may differ from country to country. Consult your specialist dealer for the guarantee period for warranty claims.

(...)

The guarantee excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the guarantee excludes parts or working parts subject to natural wear and tear.

(...)

The terms of the guarantee may differ from country to country. Please contact your dealer for further information.

3 Disposal

When disposing of the electric wheelchair or the batteries observe local regulations or contact your local disposal centre or return the product to your specialist dealer who, after submitting it to a hygienic procedure, will be able to send it back to the manufacturer who will dispose of and recycle it correctly, separating it into its component materials.

The manufacturer is responsible for taking back and recycling the electric wheelchair and shall meet the requirements of European Directive 2002/96/EC on Waste Electrical and Electronic Equipment.

Packaging materials can be taken to disposal or recycling centers or to your specialist dealer.

4 Declaration of conformity

The manufacturer or his authorized representative :

N.V. VERMEIREN N.V

Address :

Vermeirenplein 1/15
2920 Kalmthout
Belgium

declares under his sole responsibility that the CE marked devices :

Productgroup: Electric wheelchairs
Productgroup (GMDN): Wheelchair, attendant / occupant, electric drive / steer, collapsible (GMDN 40840)
FAMHP (FAGG) registration number: BE/CA01/11/2-02965-23-CLI
Brand: Vermeiren
Type: Express 2009, Express V, Express VB, Kiddie Power (Sunny), Rapido, Navix

have been classified as class I, according to annex IX MDD 93/42/EEC, rule 12, and is manufactured in full conformity with the European instructions below - including the latest modifications - and with the national law, that organizes this directions :

Medical devices directive MDD 93/42/EEC: 2007

and is in conformity with the relevant European harmonized standards:

EN 12182: 2012, EN 12184: 2014

The manufacturer or his authorized representative :

N.V. VERMEIREN N.V

Address :

Vermeirenplein 1/15
2920 Kalmthout
Belgium

declares under his sole responsibility that the CE marked devices :

Productgroup: Electric wheelchairs
Productgroup (GMDN): Wheelchair, attendant / occupant, electric drive / steer, non collapsible (GMDN 41875)
FAMHP (FAGG) registration number: BE/CA01/11/2-02965-19-CLI
Brand: Vermeiren
Type: Ascend, Forest, Forest kids, Forest narrow, Forest kids+lift (Springer), Squod, Squod SU, Tracer, Tracer 'Junior', Timix, Timix SU, Forest3, Forest3 SU, Navix SU

have been classified as class I, according to annex IX MDD 93/42/EEC, rule 12, and is manufactured in full conformity with the European instructions below - including the latest modifications - and with the national law, that organizes this directions :

Medical devices directive MDD 93/42/EEC: 2007

and is in conformity with the relevant European harmonized standards:

EN 12182: 2012, EN 12184: 2014

[illegible]

Date of the disinfection	Reason	Specification	Substance and concentration	Signature

Abbreviations used in column 2 (reason):

V = Suspected infection IF = Infection case W = Repetition I = Inspection

SERVICE

The electric wheelchair was serviced:

Dealer's stamp:

Date:

Dealer's stamp:

Date:

Dealer's stamp:

Date:

Dealer's stamp:

Date:

Dealer's stamp:

Date:

Dealer's stamp:

Date:

Dealer's stamp:

Date:

Dealer's stamp:

Date:

Dealer's stamp:

Date:

Dealer's stamp:

Date:

- For service checklists and additional technical information, please see our specialist dealers nearest to you. More information on our website at: www.vermeiren.com.

Notes

A series of horizontal dotted lines for taking notes.

Notes

A series of horizontal dotted lines for taking notes.

Belgium

N.V. Vermeiren N.V.

Vermeirenplein 1 / 15
B-2920 Kalmthout
Tel: +32(0)3 620 20 20
Fax: +32(0)3 666 48 94
website: www.vermeiren.be
e-mail: info@vermeiren.be

Germany

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Wahlerstraße 12 a
D-40472 Düsseldorf
Tel: +49(0)211 94 27 90
Fax: +49(0)211 65 36 00
website: www.vermeiren.de
e-mail: info@vermeiren.de

France

Vermeiren France S.A.

Z. I., 5, Rue d'Ennevelin
F-59710 Avelin
Tel: +33(0)3 28 55 07 98
Fax: +33(0)3 20 90 28 89
website: www.vermeiren.fr
e-mail: info@vermeiren.fr

Austria

Vermeiren Austria GmbH

Schärdinger Strasse 4
A-4061 Pasching
Tel: +43(0)7229 64900
Fax: +43(0)7229 64900-90
website: www.vermeiren.at
e-mail: info@vermeiren.at

Italy

Vermeiren Italia

Viale delle Industrie 5
I-20020 Arese MI
Tel: +39 02 99 77 07
Fax: +39 02 93 58 56 17
website: www.vermeiren.it
e-mail: info@vermeiren.it

Switzerland

Vermeiren Suisse S.A.

Hühnerhubelstraße 59
CH-3123 Belp
Tel: +41(0)31 818 40 95
Fax: +41(0)31 818 40 98
website: www.vermeiren.ch
e-mail: info@vermeiren.ch

Poland

Vermeiren Polska Sp. z o.o

ul. Łączna 1
PL-55-100 Trzebnica
Tel: +48(0)71 387 42 00
Fax: +48(0)71 387 05 74
website: www.vermeiren.pl
e-mail: info@vermeiren.pl

Spain / Portugal

Vermeiren Iberica, S.L.

Carratera de Cartellà, Km 0,5
Sant Gregori Parc Industrial Edifici A
17150 Sant Gregori (Girona)
Tel: +34 972 42 84 33
Fax: +34 972 40 50 54
website: www.vermeiren.es
e-mail: info@vermeiren.es

Czech Republic

Vermeiren ČR S.R.O.

Nádražní 132
702 00 Ostrava 1
Tel: +420 596 133 923
Fax: +420 596 121 976
website: www.vermeiren.cz
e-mail: info@vermeiren.cz

VERMEIREN

LE controller

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

EN	Contents	
1.	Use	2
	Turn wheelchair On/Off	3
	Sleep mode	3
	Using the battery charge indicator	3
	Battery charging	4
	Adjusting the driving speed	5
	Using the horn	5
	The status indicator	5
	Using the joystick	5
2.	Troubleshooting	6

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
OPERATOR CONTROL LiNX LE

1. Use

The operator control built into your electric wheelchair enables you to control all the driving, steering, braking and operating processes. The wheelchair's electrical unit and electronics are constantly being monitored internally. Any fault in the electronics is indicated on the status indicator (1) (see the chapter on troubleshooting).

Vermeiren is responsible for the changes in the software. For changes in the software contact Vermeiren.

⚠ WARNING: Risk of burns – Be aware that the surface of the operator control can potentially get hot when it is exposed to strong sunlight for long periods.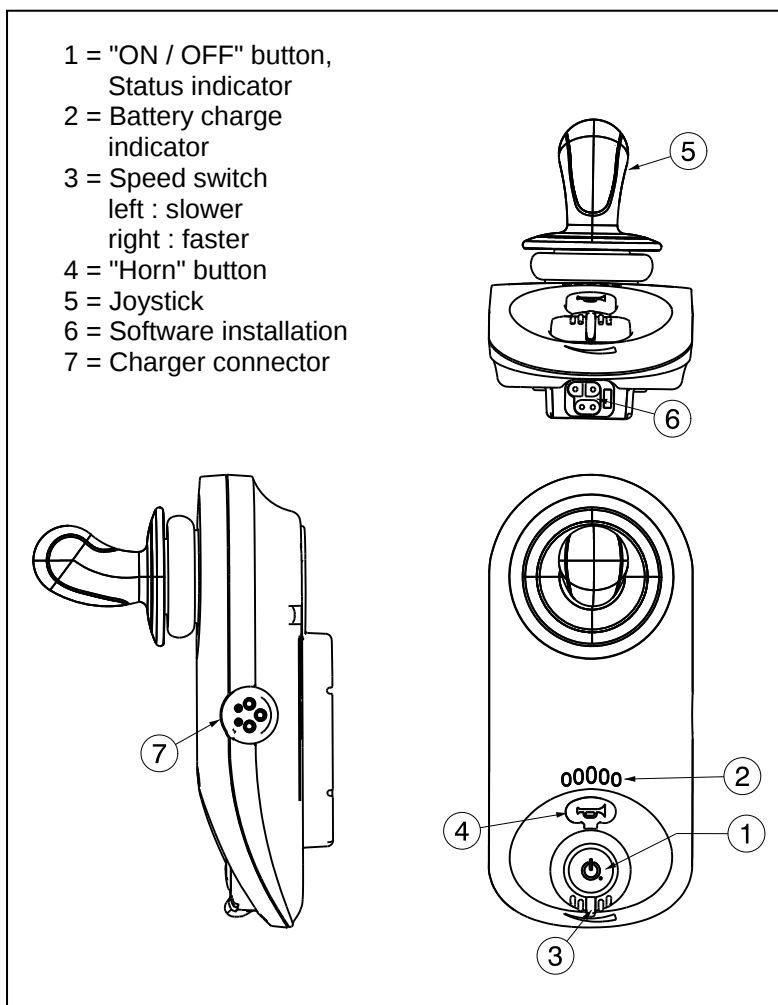

Press the "ON/OFF" button (1). The charge indicator (2), which also indicates the batteries' charge level, lights up for a moment. If all the lights come on, the batteries are sufficiently charged. If the lights do not all come on, you must adjust your driving activities to suit the low capacity of the batteries. If the battery charger display lights do not come on, check the plug connections of the electronic system.

Now select the driving program you want by turning the speed switch (3), left turn to go slower and right turn to go faster.

To set the wheelchair in motion, push the joystick forwards slowly. The speed will increase when you push the joystick further forwards. If you want to turn left or right, simply push the joystick in the desired direction. To reverse, pull the joystick backwards from the central position.

⚠ Note that, when pressing the "ON/OFF" button, the joystick stays in the central position for at least five seconds. It is programmed this way for technical and safety reasons, to prevent the joystick being pushed at the same time as the wheelchair is being switched on. If both processes are activated at the same time, the steering unit locks itself and can only become operational again after being switched off completely.


You can sound an acoustic warning signal with the "Horn" (4) button.

To brake while driving, simply push the joystick to its central position. Move the joystick slowly for gradual braking. Simply release the lever for a quick stop; the wheelchair will then stop as quickly as possible.

Turn wheelchair On/Off

Turn the power on: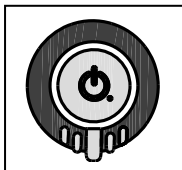


Press the On/Off button. The On/Off button will light green and all LEDs of the battery indicator turn on and display the current battery status.

If there is a fault with the system when powering up, the status indicator will indicate the fault with a series of red flashes. If the fault is one that prevents the system from driving, then the battery gauge will flash continuously.

If the wheelchair is turned on while the joystick is out of neutral, the wheelchair will not drive. Release the joystick back to neutral in 5 seconds and the fault will disappear. If the joystick is not released back to neutral (centre) within 5 seconds, the fault becomes a Module fault (the status indicator is flashing red), and the wheelchair will not drive. To clear this fault, loosen the joystick to the neutral position and turn the operator control off and then back on.

Turn the power off: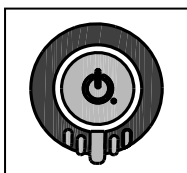

Press the On/Off button. The system will power down and the status indicator will switch off.

Emergency stop:

If the user needs to stop the wheelchair quickly, the On/Off button can be pressed to perform an EMERGENCY STOP. The wheelchair will come to a halt quickly.

Sleep mode


The operator control will go to sleep after a period of inactivity of a quarter.

When the system sleeps, it is partially turned off to reduce energy consumption.

To reactivate the operator control:

Press any button on the operator control or move the joystick. The system turns on.

Using the battery charge indicator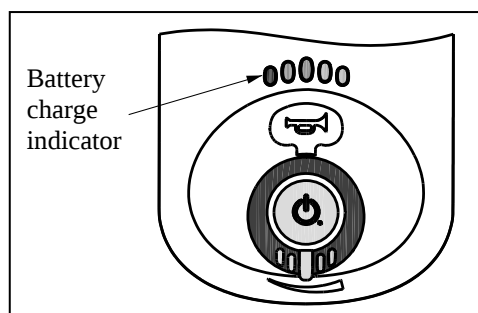

The battery charge indicator (located above the horn button) is used to indicate that your wheelchair power is on and provides an estimate of the remaining battery capacity.

Any green LED's indicate that your batteries are well charged.


If only amber and red LED's are light, the batteries are moderately charged. Recharge your batteries before you make a long trip.

If only red LED's are light, the batteries are running out of charge. Recharge as soon as possible.

The following table indicates what the battery charge indicator will display.

Display	Description	Meaning	Remarks
	All LEDs dark.	System off.	
	All LEDs are lit and the green LEDs are flashing.	System on. High voltage warning: battery voltage level has risen above the high voltage warning set-point.	Warning: battery too full. Slow down if you are driving down a slope.
	All LEDs are lit.	System on.	LEDs show the charge level.: Full battery (between 81 and 100% of full capacity)
	1 Green led is lit.	System on.	Battery level is decreasing.
	Only red and amber LEDs lit.	Low battery level.	Battery half full. Start the return journey.
	Red and amber LEDs lit.	Very low battery level.	Battery low (between 0 and 20% of full capacity). Recharge soon.
	Only Red LED lit.	Battery almost empty.	Recharge now.
	1 Red Led is flashing.	Battery empty. Low voltage warning: battery voltage level decreased below its low voltage warning set-point.	Recharge immediately.
	First red LED will flash on the battery indicator. The status indicator will display a flash code 2 or 7. The horn will sound once every 10 seconds.	Battery voltage decreases below the battery cut-off voltage.	Battery is fully discharged, and it is good practice to stop discharge at this point. The display and wheelchair shut down automatically.

Battery charging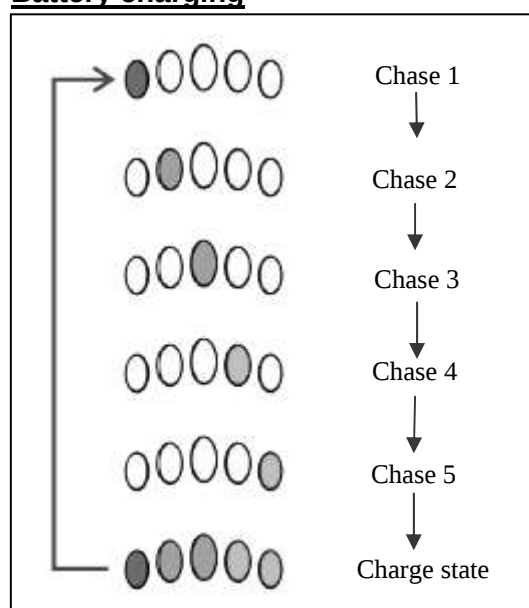

Plug the battery charger into the remote's XLR socket.


The battery charger indicator will indicate the system is being charged by cycling between a left to right chase sequence, and then displaying the approximate battery charge state at the end of the chase sequence.

Driving is prevented (inhibited) while the system is being charged.

The LE system does not have to be powered up when charging the battery, however, if it is not powered up, then the battery gauge will not display the charging state / chase sequence.

Adjusting the driving speed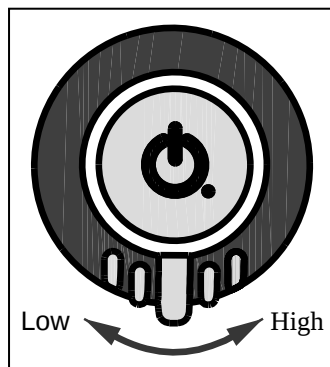

The speed switch allows the user to limit the maximum speed of the wheelchair (that is the speed when the joystick is fully deflected) their preference and environment.

The speed switch offers 10 discrete steps between the lowest speed (switch to the left) and the highest speed (switch to the right).


As a visual reminder, a speed symbol is positioned just below the speed switch to indicate the low and high positions.

Using the horn

Press the horn button (located above the ON/OFF button). The horn shall sound as long as the button is pressed.

The status indicator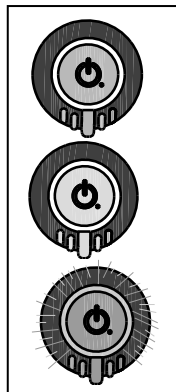

The status indicator is located below the ON/OFF button. When the LiNX LE system is not powered up, the status indicator is not lit.

When the LiNX LE system is powered up, and there are no faults with the system, the status indicator will be lit green.

If, when powered up, there is a fault with the system, then the status indicator will flash red. The number of flashes will indicate the type of error. For flash codes refer to paragraph 2: Troubleshooting.


Using the Joystick

Moving the joystick will drive the electric wheelchair in that direction. The amount of joystick movement will determine the speed that the powerchair will move in that direction.

Make sure that the joystick is in the neutral (central) position when pressing the on/off button, otherwise the electronics will be locked. To take this block away let go the joystick until he goes back in neutral position.

Drive inhibit indication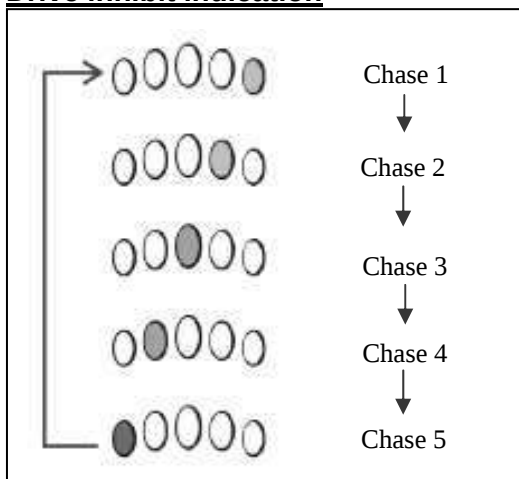


Drive inhibit mode is indicated by the battery charger indicator with a right-to-left chase sequence.

The chase sequence starts with the green LED on the right-hand side, and by one-by-one, each LED will switch on and then off. When the sequence completes at the left-most red LED, it begins again at the righthand side.

The chase sequence continues until the error condition has been cleared.

2. Troubleshooting

The following diagnoses refer to all the electronics. The various modules for steering, power have their own light signals indicating the status of the module concerned (steady light = OK; flashing light = defective). The flashing code is read from the status indicator on the operator control.

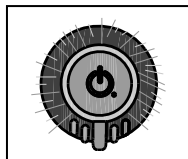If, when powered up, there is an error with the system, then the status indicator will flash red. The number of flashes will indicate the type of error. These is described in the table below.

This fault analysis aids problem analysis and problem solving by indicating the possible faults. If faults arise which cannot be identified and remedied with the aid of this analysis, kindly contact the manufacturer. We wish to point out that in any case of improper handling or if the delivered configuration has been changed, we cannot guarantee the accuracy of the fault indications.

Number	Problem/Fault	Checks
1	Remote / joystick error	1. Check that the joystick is placed in the neutral position when switch on the wheelchair. 2. Check that the joystick is not damaged and function very well.
2	Network or configuration error	1. Check all connection points, cables. 2. Verify software. 3. If you cannot found any fault contact your specialist dealer.
3	Left motor (or its connection) is defective.	1. Check whether the plugs of <u>both</u> motors have been inserted properly. 2. Check the plug contacts of <u>both</u> motors for corrosion or damage. 3. Check <u>both</u> motors. Unplug the motors and measure the plug connection with an Ohm meter. If you obtain readings of more than 1 Ohm or less than 100 milliOhms, the motor is defective. 4. Check the resistance of the motor. Measure every contact using an Ohm meter. ⚠ A Fault is also indicated if the motors are disconnected. Connect again and restart the system.
4	Right motor (or its connection) is defective.	As described above.
5	Left parking brake (or its connection) is defective or neutral active.	1. Check whether the motors' plugs have been inserted properly. 2. Check the plugs for corrosion or damage. 3. Check the parking brakes. Measure the resistance of the connections by means of an Ohm meter. If the resistance value is above 100 Ohms or beneath 20 Ohms, the parking brake is probably defective.
6	Right parking brake (or its connection) is defective.	As described above.
7	Module error (other than Remote)	One of the wheelchair parts can be damaged except the operator control itself.

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HP0180W LITHIUM BATTERY CHARGER INSTRUCTIONS

Operating Instruction:

- (1) Make sure the power cord, battery cable, battery terminals are in good condition.
- (2) Make sure the charge output voltage is the same as the connecting battery.
- (3) Connect the connector of battery charger to the connector of battery firstly, then plug the AC plug into the electricity outlet.

LED Indication:

Green light on: Power on, disconnect with the battery

Orange light on: Charging

Green light on: Fully-charged

Trouble Shooting

- (1) Green light is off when power on.
Check if the input power cord of the charger has been plug into the socket, and if there have electricity passes from the outlet, if yes; please send the charger back to the manufacture for repair
- (2) Orange light is off when connect with the battery.
Check if the charger and battery connectors are correctly connected, if both of them are in good connection, please send the battery for repair because it can not be charged.
- (3) Orange light turns to Green immediately.
Check if the battery is fully charged, if not, the charger maybe defective, please send the charger back to the manufacture for repair

Caution:

- (1) For indoor use only, do not expose to rain.
- (2) Use the charger only with less than 36V 40AH li-ion batteries.
- (3) For safety consideration, please disconnect the input side before removing the charger from the battery.
- (4) Keep the charger away from explosive gas and sparks and put it in a well ventilated area during charging.
- (5) To prolong the life of the charger, please do not connect the charger and battery for a long time after full-charged.

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